




**MOLECULAR PATHOLOGY**  
LABORATORY NETWORK, INC.

# Hematopathology Collection Kit

Client Services 1-800-932-2943

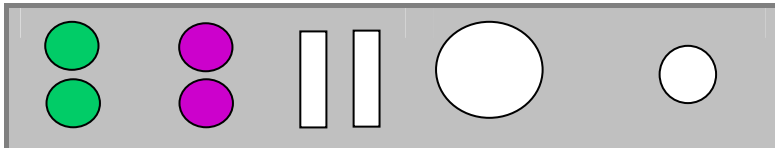

Testing Required	PB mL	BM mL	Green Top	Purple Top	Smears*	Plain tube	BM core biopsy, FNA, tissue**
Pathology	5	3		√	BM, PB	Aspirate / Clot	Formalin
Cytogenetics	7	3	√				MPLN RPMI
Fluorescence <i>in situ</i> hybridization	5	3	√				MPLN RPMI
Flow cytometry	5	3	√	√			MPLN RPMI
Gene rearrangements	5	3		√			MPLN RPMI
Quantitative PCR for MRD	10	5		√			

\* Make sure all slides are dry before placing in containers

\*\* Use Formalin or MPLN RPMI tube as necessary. MPLN RPMI should be frozen / refrigerated prior to use.

Ship overnight at room temperature. Place all specimens into the foam transport insert and overpack with the provided biohazard transport bag.

Place the sealed bag with specimens and foam into the white outer box and place appropriate paperwork inside the box. Close box and ship.

Include MPLN requisition form, a recent CBC report, and a recent clinical summary if possible.

For detailed specimen, shipping and packaging requirements refer to the test services catalogue or [www.mplnet.com](http://www.mplnet.com)